
BD 1mL Syringe

Luer-Lok™ Tip

Embout BD Luer-Lok™

Jeringa • Seringa • Seringue •
 Spritze • Siringa • Spuit • Spruta •
 Sprøjte • Ruisku • Σύριγγα •
 Sprøyte • Strzykawka • Injekcijska
 brizgalka • Injekčná striekačka •
 Süstal • Fecskendő • Švirkštas •
 Strikačka • Šlirce • Şiringa •
 Шприц • Štrcaljka • Seringă •
 Спринцовка • Шприц

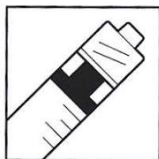REF 309628

Becton, Dickinson and Company, 1 Becton Drive
 Franklin Lakes, NJ 07417 USA

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English-en

Caution

The BD Luer-Lok™ 1mL syringe has an outer diameter equal to a BD Luer-Lok™ 3mL syringe. Syringe pumps with automatic syringe size detection will incorrectly detect this 1mL syringe as a 3mL syringe. This error will cause an under-infusion of medication by a factor of 70%.
 In order to avoid an infusion/medication error, BD® recommends that you contact the manufacturer of your syringe pump to determine if this error will occur, how to prevent it and if a revision in the programming or the software is available that will accommodate this syringe. Some manufacturers have made provisions for the BD Luer-Lok™ 1mL syringe.

Note:

Syringe pumps without automatic syringe size detection will work properly with the BD Luer-Lok™ 1mL syringe. The critical internal dimensions of the syringe that effect accuracy and performance remain unchanged from a Slip Tip 1mL syringe.
 The user enters the syringe size (1mL) into the pump without automatic syringe detection and this will allow the BD Luer-Lok™ 1mL syringe to work properly.
 Please contact the syringe pump manufacturers directly to determine the best safest method to handle this issue should you decide to use the BD Luer-Lok™ 1mL syringe in a syringe pump.

Non-Pyrogenic -

Do not use if individual package is damaged.

Not for Oral Use.

Re-use may lead to infection or other illness/injury.

Natural Rubber Latex is not part of the material formulation.

Intraocular use is not validated by BD.

DISCARD IN APPROVED SHARPS COLLECTOR AFTER SINGLE USE.

IMPORTANT SAFETY INFORMATION

To help avoid HIV (AIDS), HBV (Hepatitis) and other infectious diseases due to accidental needlesticks, used needles should not be recapped or removed, unless there is no alternative or such action is required by a specific medical procedure. OSHA (USA) standards require that recapping of needles must be performed using a one handed technique. Dispose of all needles and syringes in accordance with national/institutional guidelines.

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

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BD, the BD Logo and Luer-Lok are trademarks of
© 2021 BD. All rights reserved.