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DESCRIPTION

The bone cement mixing system is a device that ensures the correct preparation of bone cement by combining powder and liquid components, resulting in a homogenous mixture with the correct viscosity for injection.

PROCEDURAL INDICATIONS

- Before use, check the integrity of the package to guarantee sterility;
- Open the package, taking care to operate on a sterile field;
- Check that the instrument is not damaged;
- Check that the lower part is positioned upwards, if not, screw the lower body down until it is completely positioned;
- Remove the upper closing cap (1);
- Place the funnel at the mouth of the injection mould (2);
- Pour in the PMMA powder of the cement to be mixed and then the MMA monomer of the chosen cement (3);
- Remove the funnel, close the device with the upper cap by screwing it in until it is fully secured (4-5);
- Hold the device with one hand;
- Mix the cement according to the manufacturer's instructions in a twisting motion while gripping the ergonomic lever with the other hand;
- Once the cement has been mixed for the time required by the manufacturer, remove the closing cap and insert the Plunger inside the device (6-7);
- Fit the syringe to the luer-lock of the device and unscrew the lower body (8);
- Screw the Plunger in, pushing the cement inside the device downwards (9);
- Remove the quantity of product according to the manufacturer's instruction (10);
- Once the cement has been filled the syringes or injected, proceed with disposal.

LIMITATIONS AND WARNINGS

- Use allowed only by medical specialist. Tsunami Medical s.r.l. declines every responsibility for damage caused to the patient when the device is used improperly. Abnormally, as a result of incorrect operations or if used by unqualified personnel;
- The procedure described is a simple guide each physician must use the device also according to his own experience;
- Sterile, not pyrogenic, disposable device (do not reuse, re-sterilize or reprocess in any way);
- Before use, check the integrity of the packaging and the validity of the product;
- The product must not be used if the packaging is open, damaged or wet;
- Store in a cool and dry place, away from light and heat sources.

DISPOSAL INSTRUCTIONS

Used device must be dispose as "Dangerous Hospital Waste" following all applicable Laws in the country of use.

Not used device must be dispose as "Not Dangerous Hospital Waste" following all applicable Laws in the country of use.

NOTE: What above listed must be considered as a suggestion and does not exclude who discard to any responsibility related to relevant Laws.

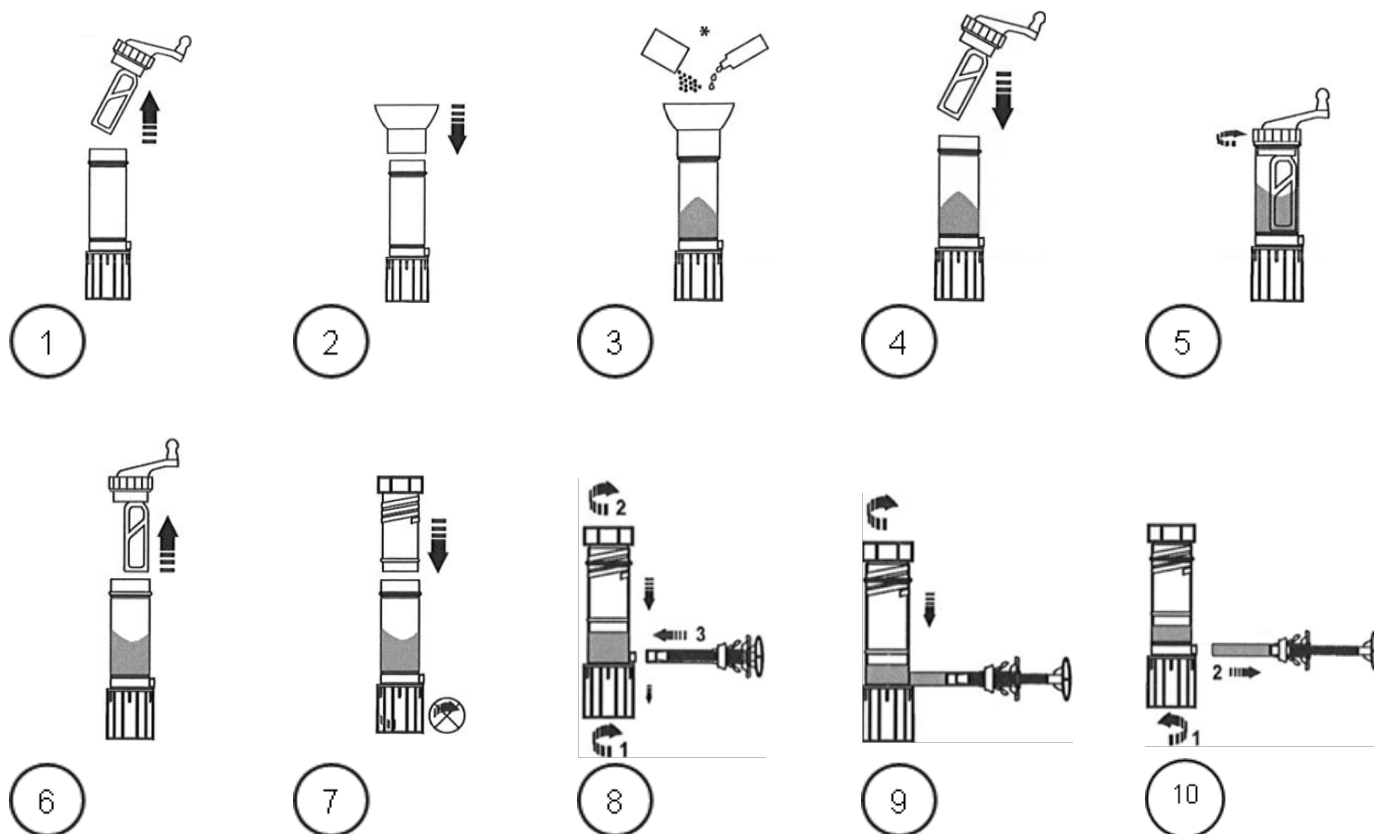
	Tsunami Medical S.r.l. Sede Legale / Registered Headquarter: Via E. Giorgi, 27 - 41124 Modena (MO) Italy Sede Produttiva / Operational Headquarter: Via XXV Aprile, 22 - 41037 Mirandola (MO) Italy Tel: +39 0535 38397 - Fax: +39 0535 38399 e-mail: info@tsunamimed.com - www.tsunamimedical.it	Page 3 of 5
--	---	-------------

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This document belong to Tsunami Medical S.r.l. Unauthorized disclosure nor copies of this document allowed.

	Tsunami Medical S.r.l. Sede Legale / Registered Headquarter: Via E. Giorgi, 27 - 41124 Modena (MO) Italy Sede Produttiva / Operational Headquarter: Via XXV Aprile, 22 - 41037 Mirandola (MO) Italy Tel: +39 0535 38397 - Fax: +39 0535 38399 e-mail: info@tsunamimed.com - www.tsunamimedical.it	Page 5 of 5
--	---	-------------

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Fabbricante: (Manufacturer Operational Headquarter): Tsunami Medical S.r.l. Via XXV Aprile, 22 41037 Mirandola (MO)